

QUICK START GUIDE English

For a perfect start.



www.miyoworld.eu



PREPARATION & INSTRUCTIONS FOR USE



Preparation

- Before applying the material, the surface must be clean and free of surface contamination.
- The surface can be sandblasted with 50 µm and 2 bar pressure.

Instructions for use

- Do not pour surplus liquid out of the container. Mix it back into a paste.
- Mix MiYO Color, MiYO Structure and Glaze Paste thoroughly with a metal free spatula before each use.
- The pastes must not come into contact with water.
- Always use a clean and dry brush.

Application

- Apply material to the prepared surface according to its purpose and required effect.

Storage and disposal

- Store in a dry place.
- Small quantities can be deposited in household waste.

**MIX MIYO COLOR, STRUCTURE AND
GLAZE PASTE THOROUGHLY WITH A
METAL-FREE SPATULA BEFORE
EACH USE.**



**Find more information in
YOUR LANGUAGE
on the USB Stick**



FIRING CHARTS & FIRING GRAPHS



Firing parameter*	COLOR STRUCTURE GLAZE PASTE FIRING			
	ZrO ₂	InSync Zr	Lithium Disilicate	Press to Zirkonia
Start temperature [°C]	400 - 450	400 - 450	400 - 450	400 - 450
Dry time [min]	3	3	3	3
Closing time [min]	4	4	4	4
Heat rate 1 and 2 [°C/min]	45	45	45	45
Holding time 1 [s]	30 - 45	30 - 45	30 - 45	30 - 45
Vacuum start [°C]	580	580	580	580
End temperature [°C]	720	720	710	720
Vacuum end [°C]	720	720	710	720
Holding time 2 [s]**	30 - 60	30 - 60	30 - 60	30 - 60
Opening time [min]	1	1	1	1

* The firing temperatures are standard values and may vary depending on the type of furnace.

** Depending on the desired shine, holding time should be 30 - 60 seconds.

HT STRUCTURE FIRING	
ZrO ₂	InSync Zr
400 - 450	400 - 450
3	3
4	4
45	45
30 - 45	30 - 45
600	600
775	775
775	775
60	60
1	1

Find more information in the customer portal:





www.miyoworld.eu/en/login-en/kundenlogin



Firing graphs

Adjustment of different firing parameters, according to the size of the restoration, for details see Workbook p.17. **Note: Delayed vacuum start!**

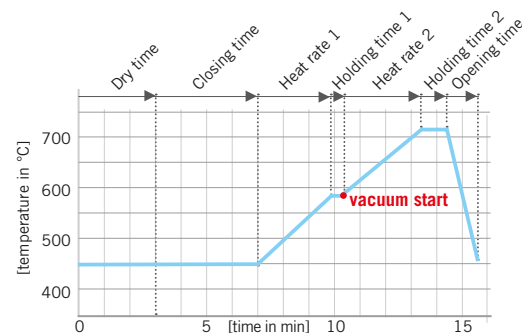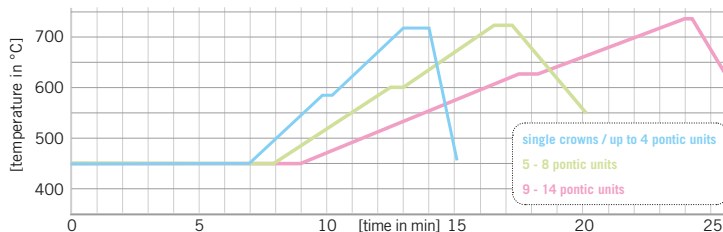

TRAINING & SUPPORT



Training videos, furnace programming ...

In the customer portal we provide you with exclusive training and information material. Learn how to use MiYO and MiYO pink safely with the help of the training videos - available in different languages. Firing instructions and notes on furnace programming help you to program your furnace.

Register: www.miyoworld.eu/en/login-en/registrierung

To register you will need the MiYO Kit number, this can be found on the back of this Quick Start Guide and in the MiYO kit under the liquid.



MiYO Online Training

Check out everything about MiYO or MiYO pink with a practical online demo. Our application technicians will give you an overview of the system, show you how to design an anterior or gingiva with MiYO and answer your questions about MiYO and MiYO pink. The online demos are suitable for newcomers and MiYO users. Register free of charge ...

www.miyoworld.eu/en/training/online



Instruction manuals

All documents concerning MiYO, MiYO pink and the glaze pastes are available in different languages for you to download from our website or on the enclosed USB stick

www.miyoworld.eu/en/downloads



Support

Do you have any questions about MiYO?
Please get in touch with us.

E-Mail: miyo@jensendental.de
Phone: +49 (0) 7123 92260
www.miyoworld.eu/en/contact



Individual training & courses

For an individual MiYO training and MiYO courses please contact your local MiYO dealer.

www.miyoworld.eu/en/dealer



PRICE LISTS & ORDER INFORMATION

MiYO

Article	Art.no.	Price €
TRANSLUCENT 4g		
Trans Shade A	0877	43,00
Trans Shade B	0878	
Trans Shade C	0879	
Trans Shade D	0880	
Trans Straw Fluor	0862	
Trans Sage Fluor	0893	
Trans Lotus Fluor	0861	
Trans Clementine Fluor	0863	
Trans Storm Fluor	0860	
Trans Cobalt Fluor	0894	
Trans Smoke Fluor	0875	
Trans Slate Fluor	0895	
Trans Lumin Fluor	0876	
Trans Lumin Plus Fluor	0896	
Trans Sunflower Fluor	0950	
Trans Garnet	0952	
MAMELON 4g		
Mamelon Wheat Fluor	0866	43,00
Mamelon Coral Fluor	0867	
Mamelon Pumpkin Fluor	0868	
HALO 4g		
Halo Spring Fluor	0864	43,00
Halo Autumn Fluor	0865	
COLOR 4g		
Snow Fluor	0857	43,00
Linen Fluor	0858	
Fissure Fluor (Pulver)	0859	

Article	Art.no.	Price €
STRUCTURE 4g		
Structure Window Fluor	0869	48,00
Structure Ghost Fluor	0873	
Structure Enamel Fluor	0872	
Structure Ice Fluor	0870	
Structure Blush Fluor	0871	
HT STRUCTURE 4g		
HT Structure Enamel Fl.	0953	48,00
HT Structure Window Fl.	0954	
GLAZE PASTE 4g		
InSync Glaze Paste Fluor	0819	39,00
STAIN / GLAZE LIQUID INSYNC one-for-all		
Stain / Glaze Liquid 25ml	0800/25	11,60
Stain / Glaze Liquid 50ml	0800/50	23,00
MiYO Kit		
18 x MiYO Color 5 x MiYO Structure 1 Glaze Paste 1 Stain / Glaze Liquid 25 ml	0881- MiYO Kit	990,00
MiYO Kit Color+		
22 x MiYO Color 2 x MiYO Structure 1 Glaze Paste 1 Stain / Glaze Liquid 25 ml	0881- MiYO Kit Color+	990,00

The price list is valid from 01.04.2021.
All prices plus VAT and shipping costs.
Errors and omissions excepted. The
General Terms and Conditions of Jensen GmbH apply.

MiYO
pink

Article	Art.no.	Price €
TRANSLUCENT 4g		
Trans Raspberry	0887	43,00
Trans Copper	0888	
Trans Midnight	0883	
Trans Carnation	0951	
Trans Garnet	0952	
COLOR 4g		
Flamingo	0881	43,00
Crimson	0882	
Plum	0884	
Merlot	0885	
Sorbet	0886	
Salmon	0889	
Sable	0897	
Thistle	0898	
Venule (Pulver)	0957	
STRUCTURE 4g		
Structure Orchid	0890	48,00
Structure Rouge	0891	
Structure Frost	0892	
GLAZE PASTE 4g		
MiYO Glaze Paste No Fluor	0821	39,00
STAIN / GLAZE LIQUID INSYNC one-for-all		
Stain / Glaze Liquid 25ml	0800/25	11,60
Stain / Glaze Liquid 50ml	0800/50	23,00
MiYO pink Kit		
11 x MiYO pink Color 3 x MiYO pink Structure 1 MiYO Glaze Paste No Fluor 1 Stain/Glaze Liquid 25 ml	0881- MiYO pink	599,00



Register for the customer portal and get more information

As a MiYO customer, you have exclusive access to the customer portal.
To do so, please register at www.miyoworld.eu under LOGIN > REGISTRATION.
To register, you need your MiYO Kit number.

You will find your individual kit number in the MiYO Kit in the lid of the packaging and under the liquid bottle.



<https://www.miyoworld.eu/en/login-en/registrierung>

Manufactured by:

Chemichl AG
Landstrasse 114
9490 Vaduz, Liechtenstein
info@chemichl.com
www.chemichl.com



Sales and Technical Support in Europe:

Jensen GmbH
Gustav-Werner-Straße 1
72555 Metzingen, Germany
Tel. +49 7123 92260
Mail: info@jensendental.de
www.miyoworld.eu

... more MiYO

